



DTX Studio™ Implant 3.6

Installation Guide

Disclaimer of liability: This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Nobel Biocare. Non-recommended use of products made by third parties in conjunction with Nobel Biocare products will void any warranty or other obligation, express or implied, of Nobel Biocare. The user of Nobel Biocare products has the duty to determine whether or not any product is suitable for the particular patient and circumstances.

Nobel Biocare disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of Nobel Biocare products. The user is also obliged to study the latest developments in regard to this Nobel Biocare product and its applications regularly. In cases of doubt, the user has to contact Nobel Biocare. Since the utilization of this product is under the control of the user, they are his/her responsibility. Nobel Biocare does not assume any liability whatsoever for damage arising thereof. Please note that some products detailed in this Installation Guide may not be regulatory cleared, released or licensed for sale in all markets.

Cybersecurity: It is advised that you have active and up-to-date antivirus and anti-malware software, together with a correctly configured firewall, on the computer on which you are using DTX Studio™ Implant. Furthermore, always lock the computer when it is left unattended.



Manufacturer:

Nobel Biocare AB,
Box 5190, 402 26
Västra Hamngatan 1,
411 17 Göteborg, Sweden
Phone: +46 31 81 88 00. Fax: +46 31 16 31 52.
www.nobelbiocare.com



Consult instructions for use

Contents


Preparing Your Computer	4
Where to Find the Installer and the Install Key	5
Single-User License on a Windows® System	6
Installing the Single-User License	6
Technical Support	7
Practice-Setup License on a Windows® System	8
Installing the Practice-Setup License	8
Technical Support	9
Single-User License on Mac OS® X	10
Installing the Single-User License	10
Technical Support	11
Practice-Setup License on Mac OS® X	12
Installing the Practice-Setup License	12
Technical Support	13
Customer Service Worldwide	14

Preparing Your Computer


1. Check that your computer is connected to the Internet.
2. Check that your computer can open and display .pdf files (double-click on a .pdf file to test). Adobe® Reader® is an example of software typically used to open and display .pdf files.
3. Log on to your computer with a user account that has administrator rights. We recommend that all other software on your computer is closed before DTX Studio™ Implant is installed.

Where to Find the Installer and the Install Key

1. Go to go.dtxstudio.com . The portal is best viewed with Chrome, Safari and Edge browsers. Internet Explorer is currently not supported.
2. Log in with the username and password that were sent to you. If this is the first time you log in to DTX Studio™ Go, you will be prompted to change the password.
Note: If you did not receive username and password by mail, please contact customer support.
3. On the **Software** tab, click the DTX Studio™ Implant tile of the version you need.
4. To download the latest version of the selected software product, click **Mac** or **Windows**, depending on the operating system you wish to install DTX Studio™ Implant on. The installer file is downloaded automatically to your download folder.

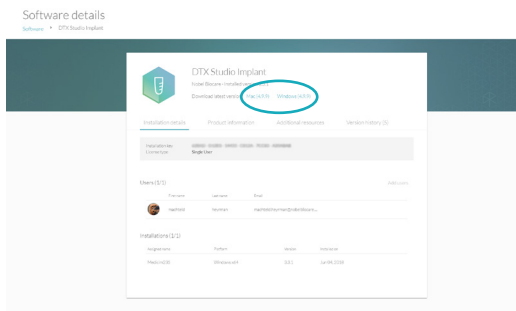

5. Copy the installation key.

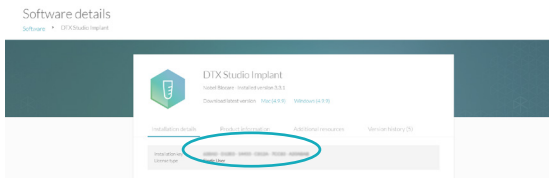

6. In this guide, go to the section of the software version you wish to install and follow the instructions:
 - Single-user license on a Windows system on page 6
 - Practice-setup license on a Windows system on page 8
 - Single-user license on Mac OS X on page 10
 - Practice-setup license on MAC OS X on page 12

Single-User License on a Windows® System

Please follow the instructions carefully.


Have the DTX Studio™ Implant license information at hand.

Note: Your license is intended for one user only. This license can be installed on two devices.

INSTALLING THE SINGLE-USER LICENSE

1. Double-click the downloaded .exe installer file.

Note: Before starting the actual installation, the system will check whether a number of prerequisite programs are available on the system you are using. If not available yet, a message is shown and the installation of the program(s) is requested. Accept to have them installed before starting the actual installation of DTX Studio™ Implant.
2. A welcome message for the install wizard appears. Click **Next**.
3. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement**, and click **Next**. Alternatively, click **Cancel** and the installation procedure will abort.
4. Choose a destination folder where the DTX Studio™ Implant files will be saved. By default, this folder will be C:\Program Files\DTX Studio Implant. To change this destination folder, click **Change**, browse to the folder of your choice and click **OK**.
5. Click **Next**.
6. Click **Install** to start the installation.
7. When the software is installed, click **Finish**. DTX Studio™ Implant starts and a configuration wizard is shown, allowing you to set up the software.
8. Click **Next**.
9. Enter the installation key that you copied from DTX Studio™ Go.
10. Click **Next**.
11. Select the folder where the patient-related data will be stored. The default location is C:\DTX Studio\Implant. Please make sure that this folder is stored on a hard disk with sufficient free disk space (approximately 250 MB per patient file is needed). All users of the computer should have read/write access to this folder.
12. Click **Next**.
13. Decide whether you would like to install products from a local file or download them from the server (a stable, high-speed Internet connection is required).
 - If you choose the local file, browse to the implant library file (with the extension .ilf) and click **Open** to install all products in the library. The installation progress is shown.
 - If you choose to download the products from the server, select **Install through the download server** and click **Next**. A new wizard page allows you to select the manufacturer(s) whose products are to be downloaded. Select the manufacturer(s) and click **Next** to start downloading. The download progress is shown. It is possible to cancel the process by returning to the previous page and switching to another option.
14. When all products are installed, click **Finish**.
15. Log on as a specific user.
16. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement**, and click **Next**. Alternatively, click **Cancel** and the installation procedure will abort.
17. When the account is successfully registered, click **Finish**. The Instructions for Use website is displayed. The software is ready to use.

A DTX Studio™ Implant icon  is available on your desktop. Double-click it to start the software. DTX Studio™ Implant can also be started by clicking the Windows® button. Browse the list of all programs that appears and click **DTX Studio™ Implant** to start the software. The Instructions for Use are also available here in a printable format.

At all times, the license agreement is available in the **Manage account** dialog within DTX Studio™ Implant. It is advised to use the software while connected to the Internet in order to receive software updates, etc. An Internet connection is required to prepare and send orders.

TECHNICAL SUPPORT

If you encounter any problems during the installation, please contact your local customer support. See page 14 for contact information.

Practice-Setup License on a Windows® System

Please follow the instructions carefully.

Have the DTX Studio™ Implant license information document at hand.

Note: For this license, a maximum of ten users can be created. This license can be installed on six devices.

INSTALLING THE PRACTICE-SETUP LICENSE


1. Double-click DTXImplant.exe.

Note: Before starting the actual installation, the system will check whether a number of prerequisite programs are available on the system you are using. If not available yet, a message is shown and the installation of the program(s) is requested. Accept to have them installed before starting the actual installation of DTX Studio™ Implant.

2. A welcome message for the install wizard appears. Click **Next**.
3. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement**, and click **Next**. Alternatively, click **Cancel** and the installation procedure will abort.
4. Choose a destination folder where the DTX Studio™ Implant files will be stored. By default, this folder will be C:\Program Files\DTX Studio Implant. To change this destination folder, click **Change**, browse to the folder of your choice and click **OK**.
5. Click **Next**.
6. Click **Install** to start the installation.
7. When the software is installed, click **Finish**. DTX Studio™ Implant starts and a configuration wizard is shown, allowing you to set up the software.
8. Click **Next**.
9. Enter the installation key that you copied from DTX Studio™ Go.
10. Click **Next**.
11. Configure the software. Select the team folder and the working folder. In the **Team folder** all patient records will be stored. It should be accessible to all practice members. You can still change this setting by selecting **General Preferences** from the **DTX Studio Implant** menu. In the **Working folder** the privately stored patient-related data will be saved. The default location is C:\DTX Studio\Implant. The location for this working folder cannot be changed in the **General Preferences**. Please make sure that this folder is stored on a hard disk with sufficient free disk space (approximately 250 MB per patient file is needed). All users of the computer should have read/write access to this folder.
12. Click **Next**.

Note: If there are Mac® and Windows® users in your practice, the team folder has to be set once on a Mac® computer for all Mac® users and once on a Windows® computer for all Windows® users.
13. Decide whether you would like to install products from a local file or download them from the server (a stable, high-speed Internet connection is required).
 - If you choose the local file, browse to the implant library file (with the extension .ilf) and click **Open** to install all products in the library. The installation progress is shown.
 - If you choose to download the products from the server, select **Install through the download server** and click **Next**. A new wizard page allows you to select the manufacturer(s) whose products are to be downloaded. Select the manufacturer(s) and click **Next** to start downloading. The download progress is shown. It is possible to cancel the process by returning to the previous page and switching to another option.
14. When all products are installed, click **Finish**.

15. Log on as a specific user.
16. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement**, and click **Next**. Alternatively, click **Cancel** and the installation procedure will abort.
17. When the account is successfully registered, click **Finish**. The Instructions for Use website is displayed. The software is ready to use.

A DTX Studio™ Implant icon  is available on your desktop. Double-click it to start the software. DTX Studio™ Implant can also be started by clicking the Windows® button. Browse the list of all programs that appears and click **Nobel Biocare**. Click **DTX Studio™ Implant** to start the software. The Instructions for Use are also available here in a printable format.

At all times, the License Agreement is available in the **Manage account** dialog within DTX Studio™ Implant. It is advised to use the software while connected to the Internet in order to receive software updates etc. An Internet connection is required to prepare and send orders.

TECHNICAL SUPPORT

If you encounter any problems during the installation, please contact your local customer support. See page 14 for contact information.

Single-User License on Mac OS® X


Please follow the instructions carefully.

Have the DTX Studio™ Implant license information document at hand.

Note: Your license is intended for one user only. This license can be installed on two devices.

INSTALLING THE SINGLE-USER LICENSE

1. Double-click the Apple® Disk Image file (.dmg) to mount it. The mounted **DTX Studio™ Implant** disk will appear on your desktop.
2. Double-click this file. An installation dialog appears, showing different options:
 - **Install DTX Studio Implant**
 - **Software License Agreement**
3. To install the software, double-click **Install DTX Studio Implant**. A welcome screen is displayed. Click **Continue**.
4. The End-User License Agreement is displayed. Read it carefully. You can also print this information.
5. Click **Continue**. A warning is displayed, asking for acceptance of the Software License Agreement.
6. Click **Agree** to continue with the installation. If you click **Disagree** the installation procedure is aborted.
7. You can keep the default location for the installation (**Applications**) or you can change it to the location of your choice. This is the disk where the software files will be saved. Click **Continue**.
8. Click **Install**.
9. Enter the username and password of an administrator account.
10. When the software is installed, click **Finish**. DTX Studio™ Implant starts and a configuration wizard is shown, allowing you to set up the software.
11. Click **Continue**.
12. Enter the installation key that is printed on the license information sheet. Click **Continue**.
13. Configure the software. Select the working folder. This is the folder where the patient-related data will be stored. The default location is \Users\Shared\DTX Studio\Implant. Please make sure that this folder is stored on a hard disk with sufficient free disk space (approximately 250 MB per patient file is needed). All users of the computer should have read/write access to this folder. Click **Continue**.
14. Decide whether you would like to install products from a local file or download them from the server (a stable, high-speed Internet connection is required).
 - If you choose the local file, browse to the implant library file (with the extension .ilf) and click **Open** to install all products in the library. The installation progress is shown.
 - Should you choose to download the products from the server, a new wizard page allows you to select the manufacturer(s) whose products are to be downloaded. Click **Continue** to start downloading. The download progress is shown. It is possible to cancel the process by returning to the previous page and switching to another option.
15. When all products are installed, click **Finish**.
16. Log on as a specific user.
17. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement** and click **Next**. Alternatively, click **Exit** and the installation procedure will abort.
18. Click **Done**. The Instructions for Use website is displayed. The software is ready to use.

A DTX Studio™ Implant icon  is available in **Applications** (default location). Click it to start the software. At all times, the License Agreement is available in the **Manage account** dialog within DTX Studio™ Implant. It is advised to use the software while connected to the Internet in order to receive software updates etc. An Internet connection is required to prepare and send orders.

TECHNICAL SUPPORT

If you encounter any problems during the installation, please contact your local customer support. See page 14 for contact information.

Practice-Setup License on Mac OS® X

Please follow the instructions carefully.

Have the DTX Studio™ Implant license information document at hand.


Notes: For this license, a maximum of ten users can be created. This license can be installed on six devices.

INSTALLING THE PRACTICE-SETUP LICENSE

1. Double-click the Apple® Disk Image file (.dmg) to mount it. The mounted **DTX Studio™ Implant** disk will appear on your desktop.
2. Double-click this file. An installation dialog appears, showing different options:
 - **Install DTX Studio™ Implant**
 - **Software License Agreement**
3. To install DTX Studio™ Implant, double-click **Install DTX Studio™ Implant**. A welcome screen is displayed. Click **Continue**.
4. The End-User License Agreement is displayed. Read it carefully. You can also print this information.
5. Click **Continue**. A warning is displayed, asking for acceptance of the Software License Agreement.
6. Click **Agree** to continue with the installation. If you click **Disagree** the installation procedure is aborted.
7. Keep the default location for the installation (**Applications**) or change it to the location of your choice. This is the disk where the software files will be saved. Click **Continue**.
8. Click **Install**.
9. Enter the username and password of an administrator account.
10. When the software is installed, click **Finish**. DTX Studio™ Implant starts and a configuration wizard is shown, allowing you to set up the software.
11. Click **Continue**.
12. Enter the installation key that is printed on the license information sheet. Click **Continue**.
13. Configure the software. Select the team folder and the working folder. In the **Team folder**, all patient records will be stored. It should be accessible to all practice members. You can still change this setting by selecting **General Preferences** from the **DTX Studio™ Implant** menu. In the **Working folder**, the privately stored patient-related data will be saved. The default location is \Users\Shared\DTX Studio Implant. The location for this folder cannot be changed in the **General Preferences**. Please make sure that this folder is stored on a hard disk with enough free disk space (approximately 250 MB per patient file is needed). All users of the computer should have read/write access to this folder.

Note: If there are Mac® and Windows® users in your practice, the team folder has to be set once on a Mac® computer for all Mac® users and once on a Windows® computer for all Windows® users:
14. Click **Continue**.
15. Decide whether you would like to install products from a local file or download them from the server (a stable, high-speed Internet connection is required).
 - If you choose the local file, browse to the implant library file (with the extension .ilf) and click **Open** to install all products in the library. The installation progress is shown.
 - Should you choose to download the products from the server, a new wizard page allows you to select the manufacturer(s) whose products are to be downloaded. Click **Continue** to start downloading. The download progress is shown. It is possible to cancel the process by returning to the previous page and switching to another option.
16. When all products are installed, click **Finish**.

17. Log on as a specific user.
18. The End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click **I accept the terms in the license agreement** and click **Next**. Alternatively, click **Exit** and the installation procedure will abort.
19. Click **Done**. The Instructions for Use website is displayed. The software is ready to use.

A DTX Studio™ Implant icon  is available in **Applications** (default location). Click it to start the software. At all times, the License Agreement is available in the **Manage account** dialog within DTX Studio™ Implant. It is advised to use the software while connected to the Internet in order to receive software updates etc. An Internet connection is required to prepare and send orders.

TECHNICAL SUPPORT

If you encounter any problems during the installation, please contact your local customer support. See page 14 for contact information.

Customer Service Worldwide

Europe and Russia

Austria

Nobel Biocare Austria
Phone: +43 1 892 89 90

Belgium

Nobel Biocare Belgium
Phone: +32 2 467 41 70

Denmark

Nobel Biocare Denmark
Phone: +45 39 40 48 46

Finland

Nobel Biocare Finland
Phone: +358 20 740 61 00

France

Nobel Biocare France
Phone: +33 1 49 20 00 31

Germany

Nobel Biocare Germany
Phone: +49 221 500 85 590

Hungary

Nobel Biocare Hungary
Phone: +36 1 279 33 79

Ireland

Nobel Biocare Ireland
Phone: toll free 1800 677 306

Italy

Nobel Biocare Italy
Phone: +39 039 683 61
Customer support: toll free 800 53 93 28

Lithuania

Nobel Biocare Lithuania
Phone: +370 5 268 3448

Netherlands

Nobel Biocare Netherlands
Phone: +31 30 635 4949

Norway

Nobel Biocare Norway
Phone: +47 64 95 75 55
Customer support: +47 23 24 98 30

Poland

Nobel Biocare Poland
Phone: +48 22 549 93 50
Customer support: +48 22 549 93 52

Portugal

Nobel Biocare Portugal
Phone: +351 22 374 73 50
Customer support: toll free 800 300 100

Russia

Nobel Biocare Russia
Phone: +7 495 974 77 55
Customer support: toll free 8 800 250 77 55

Spain

Nobel Biocare Spain
Phone: +34 93 508 8800
Customer support: toll free 900 850 008

Sweden

Nobel Biocare Sweden
Phone: +46 31 81 88 00
Customer support: +46 31 335 49 10

Switzerland

Nobel Biocare Switzerland
Phone: +41 800 211 424

United Kingdom

Nobel Biocare UK
Phone: +44 208 756 3300
Customer support: +44 18 95 43 06 50

North America

Canada

Nobel Biocare Canada
Phone: +1 905 762 3500
Customer support: +1 800 939 9394

USA

Nobel Biocare USA
Phone: +1 714 282 4800
Customer support: +1 800 322 5001

Central/South America

Argentina

Nobel Biocare Argentina
Phone: +54 11 4825 9696
Customer support: toll free 0800 800 66235

Brazil

Nobel Biocare Brazil
Phone: +55 11 5508 4800
Customer support: 0800 169 996

Mexico

Nobel Biocare Mexico
Phone: +52 55 524 974 60

Asia Pacific

Australia

Nobel Biocare Australia
Phone: +61 2 8064 5100
Customer support: toll free 1800 804 597

China

Nobel Biocare China
Phone: +86 21 5206 6655
Customer support: +86 21 5206 0974

Hong Kong

Nobel Biocare Hong Kong
Phone: +852 2845 1266
Customer support: +852 2823 8926

India

Nobel Biocare India
Phone: +91 22 6751 9999
Customer support: toll free 1 800 22 9998

Japan

Nobel Biocare Japan
Phone: +81 3 6717 6191

New Zealand

Nobel Biocare New Zealand
Phone: +64 0 800 441 657
Customer support: toll free 0800 441 657

Singapore

Nobel Biocare Singapore
Phone: +65 6737 7967
Customer support: +65 6737 7967

Taiwan

Nobel Biocare Taiwan
Phone: +886 2 2793 9933
Customer support: +886 080 00 779

Africa

South Africa

Nobel Biocare South Africa
Phone: +27 11 802 0112

Europe and Middle East Distributor Markets

Bulgaria, Croatia, Cyprus, Czech Republic, Greece, Iran, Jordan, Kuwait, Lebanon, Malta, Romania, Saudi Arabia, Serbia, Slovakia, Slovenia, Turkey, United Arab Emirates

Please refer to the [Nobel Biocare website](https://www.nobelbiocare.com) for the latest contact information.